

Make USB Update USB Stick Instructions

Doc date: 7/11/16

Purpose: The purpose of these instructions are to give guidance on how to properly download a treatment file, add (copy) to USB drive and add treatments to the LIGHTWAVE main unit using the USB port and USB flash drive.

Items Required: You will need the following items to perform the update.

LW DK Main Unit with USB port, Any USB flash drive, PC or Laptop (w/ USB port), Email/Internet Access and Update File: LWUpdate.upd

Instructions:

1. Locate the USB flash drive provided by LTW or any other USB flash drive available.



2. Insert USB flash drive into any USB port on your PC or Laptop.



3. Go to your email inbox and find the email with the treatment file attached named: "LWUpdate.upd"



4. Download or save this attachment to the root of your USB Drive. (COPY & PASTE or other)

5. Next remove the USB flash drive from the USB port and Insert USB flash drive into USB port located on right side of main unit.



6. Press "Update" button on login screen if available; otherwise reboot main unit to apply update.
7. If prompted that "An update is available..." or "Connect USB before..." Select OK and continue
8. Once completed the system will to restart. At this point the system has been updated and can be used as normal.

If you have any questions or need assistance with this process, please contact LTW's support staff for help.

Thank you,

LTW Support Staff

www.myLIGHTWAVE.com

866.999.6954 x 9105